

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN Ca Test Code: IL_ Ca-ph

Measure Sample Reagent Ranges Limits Calibration

Sample Type: Serum

Reaction Cycle: Standard Extended

Reporting Unit: ммоль /л * Decimal Points: 2

User Define: Conv. Factor: 0.000

Methodology: Type: End Point Rate

Measuring Point: 16 - 33

Photometric: 1 Wavelength 2 Wavelength

Primary: 660 Secondary: 750

Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN Ca Test Code: IL_ Ca-ph

Measure Sample Reagent Ranges Limits Calibration

Sample Volume: Sample Dilution Diluent

	Sample Volume	Dilution	Diluent
1	2.0	0.0	0.0
2	2.0	0.0	0.0
3	3.0	0.0	0.0
4	0.0	0.0	0.0

Diluent Warning Limit: 0 Tests

Reagent Volume: R1 Ca-p-1 R2 Ca-p-2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	200	0.0	✓	20	No Control
R2	50	0.0	✓	20	No Control

Sampling Condition:

Condition No.	1	2	3	4
First Run	✓			
Samp.Vol.Reduction		✓		
Below N-Range				
Above N-Range				
Panic L				
Panic H		✓		
User Range L				
User Range H	✓	✓		
Noise				
Prozone				
HIGH!	✓	✓		
ABS!	✓	✓		

Ranges		Limits																																														
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>2.15</td> <td>2.57</td> </tr> <tr> <td>Female</td> <td>2.15</td> <td>2.57</td> </tr> <tr> <td>Other</td> <td>2.15</td> <td>2.57</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	2.15	2.57	Female	2.15	2.57	Other	2.15	2.57	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td colspan="2"> <input type="radio"/> Negative <input checked="" type="radio"/> Positive </td> <td colspan="2"> <input checked="" type="radio"/> Above <input type="radio"/> Below </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td colspan="2"> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table></td></tr></tbody> </table>		Reaction Slope		Absorbance Limit		<input type="radio"/> Negative <input checked="" type="radio"/> Positive		<input checked="" type="radio"/> Above <input type="radio"/> Below		<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table>		Non-Linear Limit		0	%	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td colspan="2"> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>	Judge Point		0	
Normal Range																																																
	Lower	Upper																																														
Male	2.15	2.57																																														
Female	2.15	2.57																																														
Other	2.15	2.57																																														
Reaction Slope		Absorbance Limit																																														
<input type="radio"/> Negative <input checked="" type="radio"/> Positive		<input checked="" type="radio"/> Above <input type="radio"/> Below																																														
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table>		Non-Linear Limit		0	%	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td colspan="2"> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>	Judge Point		0																												
Non-Linear Limit																																																
0	%																																															
Prozone Limit																																																
<input type="radio"/> Above <input checked="" type="radio"/> Below																																																
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>	Judge Point		0																																								
Limit	Equation																																															
0.0	none																																															
Judge Point																																																
0																																																
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>6.2</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.01	6.2	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																													
User Range																																																
Lower	Upper																																															
0.01	6.2																																															
Serum Index Limits																																																
	Limit																																															
Hemolysis	0.0																																															
Icterus	0.0																																															
Lipemia	0.0																																															
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>6.2</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.01	6.2																																									
ValidRange																																																
Lower	Upper																																															
0.01	6.2																																															
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>				Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off																																									
Qualitative																																																
<input type="radio"/> On	<input checked="" type="radio"/> Off																																															

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																																									
NN	Ca	IL	Ca-ph																																																																
Calibration				Condition				Calibration/R-Blank Limit																																																											
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td colspan="2">1-point</td> <td colspan="2">Linear</td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Repeats</th> </tr> <tr> <td>2</td> <td></td> </tr> </thead></table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Stability</th> </tr> <tr> <td></td> <td>days</td> </tr> </thead></table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> <tr> <td></td> <td></td> </tr> </thead></table> </td> <td colspan="2"></td> </tr> </tbody> </table>				Method		Curve Type		1-point		Linear		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> </tr> <tr> <td>2</td> <td></td> </tr> </thead></table>		Repeats		2		<table border="1"> <thead> <tr> <th colspan="2">Stability</th> </tr> <tr> <td></td> <td>days</td> </tr> </thead></table>		Stability			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> <tr> <td></td> <td></td> </tr> </thead></table>		No						<input checked="" type="checkbox"/> Reagent Blank <input type="checkbox"/> Auto Reagent Blank by Bottle <input type="checkbox"/> Ask for calibration when reagent lot changes				<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td>mAbs</td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>20.0</td> <td>%</td> </tr> </thead></table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>100.0</td> <td>mAbs</td> </tr> </thead></table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>20</td> <td>%</td> </tr> </thead></table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead></table> </td> </tr> </tbody> </table>				R-Blank Limit		2500.0	mAbs	<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>20.0</td> <td>%</td> </tr> </thead></table>		Cal Reps Range		20.0	%	<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>100.0</td> <td>mAbs</td> </tr> </thead></table>		Min Cal Reps		100.0	mAbs	<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>20</td> <td>%</td> </tr> </thead></table>		Factor Change		20	%	<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead></table>		M-Point Curve Fit		0.0	%
Method		Curve Type																																																																	
1-point		Linear																																																																	
<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> </tr> <tr> <td>2</td> <td></td> </tr> </thead></table>		Repeats		2		<table border="1"> <thead> <tr> <th colspan="2">Stability</th> </tr> <tr> <td></td> <td>days</td> </tr> </thead></table>		Stability			days																																																								
Repeats																																																																			
2																																																																			
Stability																																																																			
	days																																																																		
<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> <tr> <td></td> <td></td> </tr> </thead></table>		No																																																																	
No																																																																			
R-Blank Limit																																																																			
2500.0	mAbs																																																																		
<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>20.0</td> <td>%</td> </tr> </thead></table>		Cal Reps Range		20.0	%																																																														
Cal Reps Range																																																																			
20.0	%																																																																		
<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>100.0</td> <td>mAbs</td> </tr> </thead></table>		Min Cal Reps		100.0	mAbs																																																														
Min Cal Reps																																																																			
100.0	mAbs																																																																		
<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>20</td> <td>%</td> </tr> </thead></table>		Factor Change		20	%																																																														
Factor Change																																																																			
20	%																																																																		
<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead></table>		M-Point Curve Fit		0.0	%																																																														
M-Point Curve Fit																																																																			
0.0	%																																																																		
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCalU</td> <td>**</td> </tr> <tr> <td>2</td> <td></td> <td></td> </tr> <tr> <td>3</td> <td></td> <td></td> </tr> <tr> <td>4</td> <td></td> <td></td> </tr> <tr> <td>5</td> <td></td> <td></td> </tr> </tbody> </table>				Calibrator				Calibrator	Conc	1	TruCalU	**	2			3			4			5																																													
Calibrator																																																																			
	Calibrator	Conc																																																																	
1	TruCalU	**																																																																	
2																																																																			
3																																																																			
4																																																																			
5																																																																			

* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) . Контроль по TruLab N и TruLab P.