

Холестерин

GHOD-PAP (холестериноксидазный метод)

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **Chol** Test Code: IL_ **Chol-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Type
Serum

Reaction Cycle
☒ Standard ☐ Extended

Reporting Unit
☒ ммоль/л * **Decimal Points** 2
☐ User Define
Conv.Factor 0.000

Methodology
Type
☒ End Point ☐ Rate
Measuring Point 33 -
Photometric
☐ 1 Wavelength ☒ 2 Wavelength
Primary 510 **Secondary** 700
Correction Constant
Slope 1.000 **Intercept** 0.000

Test Name: NN **Chol** Test Code: IL_ **Chol-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Volume

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	2.0	0.0	0.0
2	2.0	20.0	80.0
3	4.0	0.0	0.0
4			

Diluent
Diluent Warning Limit 0 Tests

Reagent Volume
R1 **Chol-d** **R2**

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	200	0.0	<input checked="" type="checkbox"/>	20	No Control
R2			<input type="checkbox"/>		

Sampling Condition

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Холестерин

GHOD-PAP (холестериноксидазный метод)

Ranges		Limits																																									
<table border="1"><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>0.0</td><td>5.2</td></tr><tr><td>Female</td><td>0.0</td><td>5.2</td></tr><tr><td>Other</td><td>0.0</td><td>5.2</td></tr></tbody></table>		Normal Range				Lower	Upper	Male	0.0	5.2	Female	0.0	5.2	Other	0.0	5.2	<table border="1"><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input type="radio"/> Negative <input checked="" type="radio"/> Positive</td><td><input checked="" type="radio"/> Above <input type="radio"/> Below</td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table></td><td><table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table></td></tr></tbody></table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none	<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0		
Normal Range																																											
	Lower	Upper																																									
Male	0.0	5.2																																									
Female	0.0	5.2																																									
Other	0.0	5.2																																									
Reaction Slope	Absorbance Limit																																										
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																										
<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none	<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0																									
Non-Linear Limit																																											
0	%																																										
Prozone Limit																																											
<input type="radio"/> Above <input checked="" type="radio"/> Below																																											
<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none																																							
Limit	Equation																																										
0.0	none																																										
<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0																																								
Judge Point																																											
0																																											

	User Range			------------	-------		Lower	Upper		0.08	19.4				
	ValidRange			------------	-------		Lower	Upper		0.08	19.4				
Qualitative															
☐ On ☒ Off															

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																									
NN	Chol	IL	Chol-d																																
Calibration				Condition				Calibration/R-Blank Limit																											
<table border="1"><thead><tr><th colspan="2">Method</th></tr><tr><td>1-point</td><td></td></tr></thead><tbody></tbody></table>				Method		1-point		<table border="1"><thead><tr><th colspan="2">Curve Type</th></tr><tr><td>Linear</td><td></td></tr></thead><tbody></tbody></table>				Curve Type		Linear		<table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>				R-Blank Limit		2500.0	mAbs												
Method																																			
1-point																																			
Curve Type																																			
Linear																																			
R-Blank Limit																																			
2500.0	mAbs																																		
<table border="1"><thead><tr><th colspan="2">Repeats</th></tr><tr><td>2</td><td></td></tr></thead><tbody></tbody></table>				Repeats		2		<table border="1"><thead><tr><th colspan="2">Stability</th></tr><tr><td></td><td>days</td></tr></thead><tbody></tbody></table>				Stability			days	<table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>20.0</td><td>%</td></tr></thead><tbody></tbody></table>				Cal Reps Range		20.0	%												
Repeats																																			
2																																			
Stability																																			
	days																																		
Cal Reps Range																																			
20.0	%																																		
<table border="1"><thead><tr><th colspan="2">No</th></tr><tr><td></td><td></td></tr></thead><tbody></tbody></table>				No				<table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>50.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>				Min Cal Reps		50.0	mAbs	<table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>20</td><td>%</td></tr></thead><tbody></tbody></table>				Factor Change		20	%												
No																																			
Min Cal Reps																																			
50.0	mAbs																																		
Factor Change																																			
20	%																																		
<table border="1"><thead><tr><th colspan="2">Calibrator</th></tr><tr><th></th><th>Calibrator</th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>TruCalU</td><td>**</td></tr><tr><td>2</td><td></td><td></td></tr><tr><td>3</td><td></td><td></td></tr><tr><td>4</td><td></td><td></td></tr><tr><td>5</td><td></td><td></td></tr></tbody></table>				Calibrator			Calibrator	Conc	1	TruCalU	**	2			3			4			5			<table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead><tbody></tbody></table>				M-Point Curve Fit		0.0	%				
Calibrator																																			
	Calibrator	Conc																																	
1	TruCalU	**																																	
2																																			
3																																			
4																																			
5																																			
M-Point Curve Fit																																			
0.0	%																																		
<table border="1"><thead><tr><th colspan="2">Condition</th></tr><tr><td><input checked="" type="checkbox"/> Reagent Blank</td><td></td></tr><tr><td><input type="checkbox"/> Auto Reagent Blank by Bottle</td><td></td></tr><tr><td><input type="checkbox"/> Ask for calibration when reagent lot changes</td><td></td></tr></thead><tbody></tbody></table>				Condition		<input checked="" type="checkbox"/> Reagent Blank		<input type="checkbox"/> Auto Reagent Blank by Bottle		<input type="checkbox"/> Ask for calibration when reagent lot changes																									
Condition																																			
<input checked="" type="checkbox"/> Reagent Blank																																			
<input type="checkbox"/> Auto Reagent Blank by Bottle																																			
<input type="checkbox"/> Ask for calibration when reagent lot changes																																			

Холестерин

GHOD-PAP (холестериноксидазный метод)



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) или по стандарту из набора.
Контроль по TruLab N и TruLab P.