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Test Name: NN Fe Test Code: IL\_ Fe-d

Measure Sample Reagent Ranges Limits Calibration

Sample Type: Serum

Reaction Cycle: Standard Extended

Reporting Unit: МКОЛЬ/л \* Decimal Points: 1

User Define: Conv.Factor: 0.000

Methodology: Type: End Point Rate

Measuring Point: 16 - 33

Photometric: 1 Wavelength 2 Wavelength

Primary: 600 Secondary: 750

Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN Fe Test Code: IL\_ Fe-d

Measure Sample Reagent Ranges Limits Calibration

Sample Volume: Sampling Condition

Sample	Dilution	Diluent	Condition No.	1	2	3	4
1	16.0	0.0	First Run				
2	4.0	0.0	Samp.Vol.Reduction				
3	32.0	0.0	Below N-Range				
4	0.0	0.0	Above N-Range				
			Panic L				
			Panic H				
			User Range L				
			User Range H				
			Noise				
			Prozone				
			HIGH!				
			ABS!				

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Ranges		Limits																																		
<table border="1"><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>6.3</td><td>30.1</td></tr><tr><td>Female</td><td>4.1</td><td>29.5</td></tr><tr><td>Other</td><td>4.1</td><td>30.1</td></tr></tbody></table>		Normal Range				Lower	Upper	Male	6.3	30.1	Female	4.1	29.5	Other	4.1	30.1	<table border="1"><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input type="radio"/> Negative <input checked="" type="radio"/> Positive</td><td><input checked="" type="radio"/> Above <input type="radio"/> Below</td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table></td><td><table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit: 0.0</td><td>Equation: none</td></tr><tr><td colspan="2">Judge Point: 0</td></tr></thead></table></td></tr></tbody></table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit: 0.0</td><td>Equation: none</td></tr><tr><td colspan="2">Judge Point: 0</td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		Limit: 0.0	Equation: none	Judge Point: 0	
Normal Range																																				
	Lower	Upper																																		
Male	6.3	30.1																																		
Female	4.1	29.5																																		
Other	4.1	30.1																																		
Reaction Slope	Absorbance Limit																																			
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																			
<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit: 0.0</td><td>Equation: none</td></tr><tr><td colspan="2">Judge Point: 0</td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		Limit: 0.0	Equation: none	Judge Point: 0																								
Non-Linear Limit																																				
0	%																																			
Prozone Limit																																				
<input type="radio"/> Above <input checked="" type="radio"/> Below																																				
Limit: 0.0	Equation: none																																			
Judge Point: 0																																				

	User Range			------------	-------		Lower	Upper		0.9	179				
	ValidRange			------------	-------		Lower	Upper		0.9	179				
Qualitative															
☐ On ☒ Off															

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																													
NN	Fe	IL	Fe-d																																				
Calibration				Condition				Calibration/R-Blank Limit																															
<table border="1"><thead><tr><th colspan="2">Method</th></tr><tr><td>1-point</td><td></td></tr></thead><tbody></tbody></table>				Method		1-point		<table border="1"><thead><tr><th colspan="2">Curve Type</th></tr><tr><td>Linear</td><td></td></tr></thead><tbody></tbody></table>				Curve Type		Linear		<table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>				R-Blank Limit		2500.0	mAbs																
Method																																							
1-point																																							
Curve Type																																							
Linear																																							
R-Blank Limit																																							
2500.0	mAbs																																						
<table border="1"><thead><tr><th colspan="2">Repeats</th></tr><tr><td>2</td><td></td></tr></thead><tbody></tbody></table>				Repeats		2		<table border="1"><thead><tr><th colspan="2">Stability</th></tr><tr><td></td><td>days</td></tr></thead><tbody></tbody></table>				Stability			days	<table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>20.0</td><td>%</td></tr></thead><tbody></tbody></table>				Cal Reps Range		20.0	%																
Repeats																																							
2																																							
Stability																																							
	days																																						
Cal Reps Range																																							
20.0	%																																						
<table border="1"><thead><tr><th colspan="2">No</th></tr><tr><td></td><td></td></tr></thead><tbody></tbody></table>				No				<table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>50.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>				Min Cal Reps		50.0	mAbs	<table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>20</td><td>%</td></tr></thead><tbody></tbody></table>				Factor Change		20	%																
No																																							
Min Cal Reps																																							
50.0	mAbs																																						
Factor Change																																							
20	%																																						
<table border="1"><thead><tr><th colspan="2">Calibrator</th></tr><tr><th></th><th>Calibrator</th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>TruCalU</td><td>**</td></tr><tr><td>2</td><td></td><td></td></tr><tr><td>3</td><td></td><td></td></tr><tr><td>4</td><td></td><td></td></tr><tr><td>5</td><td></td><td></td></tr></tbody></table>				Calibrator			Calibrator	Conc	1	TruCalU	**	2			3			4			5			<table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead><tbody></tbody></table>				M-Point Curve Fit		0.0	%	<table border="1"><thead><tr><th colspan="2">Reagent Blank</th></tr><tr><td><input checked="" type="checkbox"/></td><td></td></tr></thead><tbody></tbody></table>				Reagent Blank		<input checked="" type="checkbox"/>	
Calibrator																																							
	Calibrator	Conc																																					
1	TruCalU	**																																					
2																																							
3																																							
4																																							
5																																							
M-Point Curve Fit																																							
0.0	%																																						
Reagent Blank																																							
<input checked="" type="checkbox"/>																																							
				<table border="1"><thead><tr><th colspan="2">Auto Reagent Blank by Bottle</th></tr><tr><td><input type="checkbox"/></td><td></td></tr></thead><tbody></tbody></table>				Auto Reagent Blank by Bottle		<input type="checkbox"/>		<table border="1"><thead><tr><th colspan="2">Ask for calibration when reagent lot changes</th></tr><tr><td><input type="checkbox"/></td><td></td></tr></thead><tbody></tbody></table>				Ask for calibration when reagent lot changes		<input type="checkbox"/>																					
Auto Reagent Blank by Bottle																																							
<input type="checkbox"/>																																							
Ask for calibration when reagent lot changes																																							
<input type="checkbox"/>																																							

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