

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **GLDH** Test Code: IL_ **GLDH-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Type
Serum

Reaction Cycle
☒ Standard ☐ Extended

Reporting Unit
☒ Ед/л * ☐ User Define
Decimal Points: 2
Conv. Factor: 0.000

Methodology
Type
☐ End Point ☒ Rate
Measuring Point
19 - ☒ 29
Photometric
☐ 1 Wavelength ☒ 2 Wavelength
Primary 340 **Secondary** 546
Correction Constant
Slope 1.000 **Intercept** 0.000

Test Name: NN **GLDH** Test Code: IL_ **GLDH-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Volume

	Sample Volume	Dilution Sample Vol.	Dilution Diluent Vol.
1	21.0	0.0	0.0
2	2.5	0.0	0.0
3	30.0	0.0	0.0
4	10.0	10.0	90.0

Diluent **Diluent Warning Limit** 0 Tests

Reagent Volume

R1: **GLDH-d1** R2: **GLDH-d2**

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	140	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	35	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

Condition No.	1	2	3	4
First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
User Range H	<input checked="" type="checkbox"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
HIGH!	<input checked="" type="checkbox"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
ABS!	<input checked="" type="checkbox"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

ГЛДГ (глутаматдегидрогеназа, GLDH) DGKC-1970

Ranges		Limits																																																					
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>3.2</td> <td>7.3</td> </tr> <tr> <td>Female</td> <td>2.6</td> <td>6.7</td> </tr> <tr> <td>Other</td> <td>2.6</td> <td>7.3</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	3.2	7.3	Female	2.6	6.7	Other	2.6	7.3	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input checked="" type="radio"/> Negative <input type="radio"/> Positive </td> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>20</td> <td>%</td> </tr> </thead></table> </td> <td> <table border="1"> <tr> <td>700.0</td> <td>mAbs</td> </tr> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above <input checked="" type="radio"/> Below</td> <td></td> </tr> <tr> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> <td> <table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table> </td> </tr> </thead></table></td></tr></tbody> </table>		Reaction Slope	Absorbance Limit	<input checked="" type="radio"/> Negative <input type="radio"/> Positive	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>20</td> <td>%</td> </tr> </thead></table>	Non-Linear Limit		20	%	<table border="1"> <tr> <td>700.0</td> <td>mAbs</td> </tr> </table>	700.0	mAbs	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above <input checked="" type="radio"/> Below</td> <td></td> </tr> <tr> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> <td> <table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table> </td> </tr> </thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none	<table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table>	Judge Point	0
Normal Range																																																							
	Lower	Upper																																																					
Male	3.2	7.3																																																					
Female	2.6	6.7																																																					
Other	2.6	7.3																																																					
Reaction Slope	Absorbance Limit																																																						
<input checked="" type="radio"/> Negative <input type="radio"/> Positive	<input type="radio"/> Above <input checked="" type="radio"/> Below																																																						
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>20</td> <td>%</td> </tr> </thead></table>	Non-Linear Limit		20	%	<table border="1"> <tr> <td>700.0</td> <td>mAbs</td> </tr> </table>	700.0	mAbs																																																
Non-Linear Limit																																																							
20	%																																																						
700.0	mAbs																																																						
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above <input checked="" type="radio"/> Below</td> <td></td> </tr> <tr> <td> <table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table> </td> <td> <table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table> </td> </tr> </thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none	<table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table>	Judge Point	0																															
Serum Index Limits																																																							
	Limit																																																						
Hemolysis	0.0																																																						
Icterus	0.0																																																						
Lipemia	0.0																																																						
Prozone Limit																																																							
<input type="radio"/> Above <input checked="" type="radio"/> Below																																																							
<table border="1"> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> </table>	Limit	Equation	0.0	none	<table border="1"> <tr> <td>Judge Point</td> </tr> <tr> <td>0</td> </tr> </table>	Judge Point	0																																																
Limit	Equation																																																						
0.0	none																																																						
Judge Point																																																							
0																																																							
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2.0</td> <td>120</td> </tr> </tbody> </table>		User Range		Lower	Upper	2.0	120																																																
User Range																																																							
Lower	Upper																																																						
2.0	120																																																						
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2.0</td> <td>120</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	2.0	120																																																
ValidRange																																																							
Lower	Upper																																																						
2.0	120																																																						
Qualitative <input type="radio"/> On <input checked="" type="radio"/> Off																																																							

Test Name		Test Code		Measure	Sample Reagent	Ranges Limits	Calibration																												
NN	GLDH	IL_	GLDH-d																																
Calibration		Condition			Calibration/R-Blank Limit																														
<table border="1"> <thead> <tr> <th>Method</th> <th>Curve Type</th> </tr> </thead> <tbody> <tr> <td>1-point</td> <td>Linear</td> </tr> <tr> <td>Repeats: 2</td> <td>Stability: days</td> </tr> <tr> <td>No</td> <td></td> </tr> </tbody> </table>		Method	Curve Type	1-point	Linear	Repeats: 2	Stability: days	No		<input checked="" type="checkbox"/> Reagent Blank <input type="checkbox"/> Auto Reagent Blank by Bottle <input type="checkbox"/> Ask for calibration when reagent lot changes			<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <td>2500.0</td> <td>mAbs</td> </tr> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>20.0</td> <td>%</td> </tr> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>50.0</td> <td>mAbs</td> </tr> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>20</td> <td>%</td> </tr> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead></table>			R-Blank Limit		2500.0	mAbs	Cal Reps Range		20.0	%	Min Cal Reps		50.0	mAbs	Factor Change		20	%	M-Point Curve Fit		0.0	%
Method	Curve Type																																		
1-point	Linear																																		
Repeats: 2	Stability: days																																		
No																																			
R-Blank Limit																																			
2500.0	mAbs																																		
Cal Reps Range																																			
20.0	%																																		
Min Cal Reps																																			
50.0	mAbs																																		
Factor Change																																			
20	%																																		
M-Point Curve Fit																																			
0.0	%																																		
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCalU</td> <td>**</td> </tr> <tr> <td>2</td> <td></td> <td></td> </tr> <tr> <td>3</td> <td></td> <td></td> </tr> <tr> <td>4</td> <td></td> <td></td> </tr> <tr> <td>5</td> <td></td> <td></td> </tr> </tbody> </table>		Calibrator				Calibrator	Conc	1	TruCalU	**	2			3			4			5															
Calibrator																																			
	Calibrator	Conc																																	
1	TruCalU	**																																	
2																																			
3																																			
4																																			
5																																			

* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) или по фактору (см. инструкцию).
Контроль по TruLab N и TruLab P.