

Гликированный гемоглобин, HbA1c

ImmunoTD (Иммунотурбидиметрический, усиленный латексом)

Параметры для ввода в программу анализатора iLab-Taurus

Test Name
NN **HbA1c**

Test Code
IL_ **HbA1c**

Measure

Sample Reagent

Ranges Limits

Calibration

Sample Type
Serum

Reaction Cycle
☒ Standard
 ☐ Extended

Reporting Unit
☒ %*

Decimal Points
 2

☐ User Define

Conv.Factor
 0.000

Methodology

Type
☐ End Point
 ☒ Rate

Measuring Point

19

 -

☒

 28

Photometric
☒ 1 Wavelength
 ☐ 2 Wavelength

Primary
660

Secondary

Correction Constant

Slope
1.000

Intercept
0.000

Test Name
NN **HbA1c**

Test Code
IL_ **HbA1c**

Measure

Sample Reagent

Ranges Limits

Calibration

Sample Volume

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	5.0	0.0	0.0
2	5.0	0.0	0.0
3	5.0	0.0	0.0
4	0.0	0.0	0.0

Diluent

Diluent Warning Limit
0 Tests

Reagent Volume

R1
HbA1c1

R2
HbA1c2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	120	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	60	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic H	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Гликированный гемоглобин, HbA1c

ImmunoTD (Иммунотурбидиметрический, усиленный латексом)

Ranges		Limits																																																																																																				
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>4.0</td> <td>6.0</td> </tr> <tr> <td>Female</td> <td>4.0</td> <td>6.0</td> </tr> <tr> <td>Other</td> <td>4.0</td> <td>6.0</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	4.0	6.0	Female	4.0	6.0	Other	4.0	6.0	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> Negative</td> <td><input checked="" type="radio"/> Positive</td> <td><input checked="" type="radio"/> Above</td> <td><input type="radio"/> Below</td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="3">User Range</th> </tr> <tr> <th>Lower</th> <th colspan="2">Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td colspan="2">20</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="3">Valid Range</th> </tr> <tr> <th>Lower</th> <th colspan="2">Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td colspan="2">20</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> </tbody> </table>		Reaction Slope		Absorbance Limit		<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		Non-Linear Limit		100	%			<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	Judge Point		0				<table border="1"> <thead> <tr> <th colspan="3">User Range</th> </tr> <tr> <th>Lower</th> <th colspan="2">Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td colspan="2">20</td> </tr> </tbody> </table>		User Range			Lower	Upper		2	20				<table border="1"> <thead> <tr> <th colspan="3">Valid Range</th> </tr> <tr> <th>Lower</th> <th colspan="2">Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td colspan="2">20</td> </tr> </tbody> </table>		Valid Range			Lower	Upper		2	20				<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0			<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off				
Normal Range																																																																																																						
	Lower	Upper																																																																																																				
Male	4.0	6.0																																																																																																				
Female	4.0	6.0																																																																																																				
Other	4.0	6.0																																																																																																				
Reaction Slope		Absorbance Limit																																																																																																				
<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below																																																																																																			
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		Non-Linear Limit		100	%			<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	Judge Point		0																																																																																
Non-Linear Limit																																																																																																						
100	%																																																																																																					
Prozone Limit																																																																																																						
<input type="radio"/> Above	<input checked="" type="radio"/> Below																																																																																																					
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	Judge Point		0																																																																																														
Limit	Equation																																																																																																					
0.0	none																																																																																																					
Judge Point																																																																																																						
0																																																																																																						
<table border="1"> <thead> <tr> <th colspan="3">User Range</th> </tr> <tr> <th>Lower</th> <th colspan="2">Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td colspan="2">20</td> </tr> </tbody> </table>		User Range			Lower	Upper		2	20																																																																																													
User Range																																																																																																						
Lower	Upper																																																																																																					
2	20																																																																																																					
<table border="1"> <thead> <tr> <th colspan="3">Valid Range</th> </tr> <tr> <th>Lower</th> <th colspan="2">Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td colspan="2">20</td> </tr> </tbody> </table>		Valid Range			Lower	Upper		2	20																																																																																													
Valid Range																																																																																																						
Lower	Upper																																																																																																					
2	20																																																																																																					
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																																											
Serum Index Limits																																																																																																						
	Limit																																																																																																					
Hemolysis	0.0																																																																																																					
Icterus	0.0																																																																																																					
Lipemia	0.0																																																																																																					
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																																	
Qualitative																																																																																																						
<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																																					

Test Name		Test Code		Measure	Sample Reagent	Ranges Limits	Calibration																																																																																	
NN	HbA1c	IL	HbA1c																																																																																					
Calibration		Condition		Calibration/R-Blank Limit																																																																																				
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>M-point</td> <td></td> <td>4P Log</td> <td></td> </tr> </tbody> </table>		Method		Curve Type		M-point		4P Log		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> </tbody> </table>		Repeats		Stability		2			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> </thead> <tbody> <tr> <td>5</td> <td></td> </tr> </tbody> </table>		No		5		<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>HemSol</td> <td>2.0</td> </tr> <tr> <td>2</td> <td>TC-HbA1c-1</td> <td>**</td> </tr> <tr> <td>3</td> <td>TC-HbA1c-2</td> <td>**</td> </tr> <tr> <td>4</td> <td>TC-HbA1c-3</td> <td>**</td> </tr> <tr> <td>5</td> <td>TC-HbA1c-4</td> <td>**</td> </tr> </tbody> </table>		Calibrator				Calibrator	Conc	1	HemSol	2.0	2	TC-HbA1c-1	**	3	TC-HbA1c-2	**	4	TC-HbA1c-3	**	5	TC-HbA1c-4	**	<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		R-Blank Limit		2500.0	mAbs			<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>50.0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		Cal Reps Range		50.0	%			<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>100.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		Min Cal Reps		100.0	mAbs			<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>35</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		Factor Change		35	%			<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		M-Point Curve Fit		0.0	%		
Method		Curve Type																																																																																						
M-point		4P Log																																																																																						
Repeats		Stability																																																																																						
2			days																																																																																					
No																																																																																								
5																																																																																								
Calibrator																																																																																								
	Calibrator	Conc																																																																																						
1	HemSol	2.0																																																																																						
2	TC-HbA1c-1	**																																																																																						
3	TC-HbA1c-2	**																																																																																						
4	TC-HbA1c-3	**																																																																																						
5	TC-HbA1c-4	**																																																																																						
R-Blank Limit																																																																																								
2500.0	mAbs																																																																																							
Cal Reps Range																																																																																								
50.0	%																																																																																							
Min Cal Reps																																																																																								
100.0	mAbs																																																																																							
Factor Change																																																																																								
35	%																																																																																							
M-Point Curve Fit																																																																																								
0.0	%																																																																																							
		<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> <tr> <td><input type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		Condition		<input type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes																																																																													
Condition																																																																																								
<input type="checkbox"/>	Reagent Blank																																																																																							
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																																																																							
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																																																																							

* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Калибровка многоточечная (5 точек, 0+4 уровня) Spline или 4P Log,
(калибратор TruCal HbA1c). Первая точка - гемолизирующий р-р.
Контроль по TruLab HbA1c уровень 1 и уровень 2.

Перед работой готовится необходимое количество смеси R2+R3 (см инструкцию)

!! Лизирующий не входит в состав набора реагентов, - покупается отдельно !!