

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **LDH** Test Code: IL_ **LDH-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Type
Serum

Reaction Cycle
☒ Standard ☐ Extended

Reporting Unit
☒ Ед/л * ☐ User Define
Decimal Points: 1
Conv.Factor: 0.000

Methodology
Type
☐ End Point ☒ Rate
Measuring Point
19 - ☒ 29
Photometric
☐ 1 Wavelength ☒ 2 Wavelength
Primary 340 **Secondary** 546
Correction Constant
Slope 1.000 **Intercept** 0.000

Test Name: NN **LDH** Test Code: IL_ **LDH-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Volume

	Sample Volume	Dilution Sample Vol.	Dilution Diluent Vol.
1	2.0	0.0	0.0
2	2.0	10.0	90.0
3	4.0	0.0	0.0
4	2.0	4.0	96.0

Diluent **Diluent Warning Limit** 0 Tests

Reagent Volume

R1 **LDH-d1** R2 **LDH-d2**

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	200	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	50	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

Condition No.	1	2	3	4
First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Ranges		Limits																																																																																		
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0</td> <td>480</td> </tr> <tr> <td>Female</td> <td>0</td> <td>480</td> </tr> <tr> <td>Other</td> <td>0</td> <td>480</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	0	480	Female	0	480	Other	0	480	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td><input checked="" type="radio"/> Negative</td> <td><input type="radio"/> Positive</td> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>25</td> <td>%</td> </tr> </thead></table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead></table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>2500</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>2500</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> </table> </td> </tr> </tbody> </table>		Reaction Slope		Absorbance Limit		<input checked="" type="radio"/> Negative	<input type="radio"/> Positive	<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>25</td> <td>%</td> </tr> </thead></table>		Non-Linear Limit		25	%	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead></table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>		Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>2500</td> </tr> </tbody> </table>		User Range		Lower	Upper	10	2500	<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>2500</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	10	2500	<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>		Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> </table>		Judge Point		0	
Normal Range																																																																																				
	Lower	Upper																																																																																		
Male	0	480																																																																																		
Female	0	480																																																																																		
Other	0	480																																																																																		
Reaction Slope		Absorbance Limit																																																																																		
<input checked="" type="radio"/> Negative	<input type="radio"/> Positive	<input type="radio"/> Above	<input checked="" type="radio"/> Below																																																																																	
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>25</td> <td>%</td> </tr> </thead></table>		Non-Linear Limit		25	%	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead></table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below																																																																									
Non-Linear Limit																																																																																				
25	%																																																																																			
Prozone Limit																																																																																				
<input type="radio"/> Above	<input checked="" type="radio"/> Below																																																																																			
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>		Limit	Equation	0.0	none																																																																			
Serum Index Limits																																																																																				
	Limit																																																																																			
Hemolysis	0.0																																																																																			
Icterus	0.0																																																																																			
Lipemia	0.0																																																																																			
Limit	Equation																																																																																			
0.0	none																																																																																			
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>2500</td> </tr> </tbody> </table>		User Range		Lower	Upper	10	2500	<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>2500</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	10	2500																																																																					
User Range																																																																																				
Lower	Upper																																																																																			
10	2500																																																																																			
ValidRange																																																																																				
Lower	Upper																																																																																			
10	2500																																																																																			
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>		Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> </table>		Judge Point		0																																																																										
Qualitative																																																																																				
<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																			
Judge Point																																																																																				
0																																																																																				

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																		
NN	LDH	IL	LDH-d																																									
Calibration				Condition				Calibration/R-Blank Limit																																				
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>1-point</td> <td></td> <td>Linear</td> <td></td> </tr> </tbody> </table>				Method		Curve Type		1-point		Linear		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> </tbody> </table>				Repeats		Stability		2			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>				No																
Method		Curve Type																																										
1-point		Linear																																										
Repeats		Stability																																										
2			days																																									
No																																												
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCalU</td> <td>**</td> </tr> <tr> <td>2</td> <td></td> <td></td> </tr> <tr> <td>3</td> <td></td> <td></td> </tr> <tr> <td>4</td> <td></td> <td></td> </tr> <tr> <td>5</td> <td></td> <td></td> </tr> </tbody> </table>				Calibrator				Calibrator	Conc	1	TruCalU	**	2			3			4			5			<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> </thead> <tbody> <tr> <td><input checked="" type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>				Condition		<input checked="" type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes	<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </tbody> </table>				R-Blank Limit		2500.0	mAbs
Calibrator																																												
	Calibrator	Conc																																										
1	TruCalU	**																																										
2																																												
3																																												
4																																												
5																																												
Condition																																												
<input checked="" type="checkbox"/>	Reagent Blank																																											
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																											
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																											
R-Blank Limit																																												
2500.0	mAbs																																											
				<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> </thead> <tbody> <tr> <td>20.0</td> <td>%</td> </tr> </tbody> </table>				Cal Reps Range		20.0	%	<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> </thead> <tbody> <tr> <td>50.0</td> <td>mAbs</td> </tr> </tbody> </table>				Min Cal Reps		50.0	mAbs																									
Cal Reps Range																																												
20.0	%																																											
Min Cal Reps																																												
50.0	mAbs																																											
				<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> </thead> <tbody> <tr> <td>20</td> <td>%</td> </tr> </tbody> </table>				Factor Change		20	%	<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>%</td> </tr> </tbody> </table>				M-Point Curve Fit		0.0	%																									
Factor Change																																												
20	%																																											
M-Point Curve Fit																																												
0.0	%																																											

ЛДГ (лактатдегидрогеназа)

DGKC-1970



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) или по фактору (см инструкцию).

Контроль по TruLab N и TruLab P.