

# Липопротейн (а) 21

ИтипоTD (Иммунотурбидиметрический)

## Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN Lpa Test Code: IL\_ Lpa-d

Measure Sample Reagent Ranges Limits Calibration

Sample Type: Serum

Reaction Cycle: Standard Extended

Reporting Unit: Мг/л\* Decimal Points: 1

User Define: Conv.Factor: 0.000

Methodology: Type: End Point Rate

Measuring Point: 19 - 33

Photometric: 1 Wavelength 2 Wavelength

Primary: 700 Secondary:

Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN Lpa Test Code: IL\_ Lpa-d

Measure Sample Reagent Ranges Limits Calibration

Sample Volume: Sampling Condition:

Sample	Dilution	Diluent
Volume	Sample Vol.	Diluent Vol.
1 3.75	0.0	0.0
2 2.0	0.0	0.0
3 5.0	0.0	0.0
4 0.0	0.0	0.0

Diluent Warning Limit: 0 Tests

Reagent Volume: R1 Lpa-1 R2 Lpa-2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	150	0.0	✓	20	No Control
R2	75	0.0	✓	20	No Control

Condition No.	1	2	3	4
First Run	✓			
Samp.Vol.Reduction		✓		
Below N-Range				
Above N-Range				
Panic L				
Panic H				
User Range L				
User Range H	✓	✓		
Noise				
Prozone				
HIGH!	✓	✓		
ABS!	✓	✓		

# Липопротеин (а) 21

ИммуноТД (Иммунотурбидиметрический)

Ranges		Limits																																																																																				
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0.0</td> <td>300</td> </tr> <tr> <td>Female</td> <td>0.0</td> <td>300</td> </tr> <tr> <td>Other</td> <td>0.0</td> <td>300</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	0.0	300	Female	0.0	300	Other	0.0	300	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> Negative</td> <td><input checked="" type="radio"/> Positive</td> <td><input checked="" type="radio"/> Above</td> <td><input type="radio"/> Below</td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table> </td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>1100</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>1100</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="2"> <b>Qualitative</b>  <input type="radio"/> On <input checked="" type="radio"/> Off             </td> <td colspan="2"></td> </tr> </tbody> </table>		Reaction Slope		Absorbance Limit		<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>		Non-Linear Limit		100	%	<input type="text"/>	<input type="text"/>	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>	Judge Point		0		<input type="text"/>	<input type="text"/>	<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>1100</td> </tr> </tbody> </table>		User Range		Lower	Upper	10	1100	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>1100</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	10	1100			<b>Qualitative</b> <input type="radio"/> On <input checked="" type="radio"/> Off			
Normal Range																																																																																						
	Lower	Upper																																																																																				
Male	0.0	300																																																																																				
Female	0.0	300																																																																																				
Other	0.0	300																																																																																				
Reaction Slope		Absorbance Limit																																																																																				
<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below																																																																																			
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>		Non-Linear Limit		100	%	<input type="text"/>	<input type="text"/>	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>	Judge Point		0		<input type="text"/>	<input type="text"/>																																																													
Non-Linear Limit																																																																																						
100	%																																																																																					
<input type="text"/>	<input type="text"/>																																																																																					
Prozone Limit																																																																																						
<input type="radio"/> Above	<input checked="" type="radio"/> Below																																																																																					
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>	Judge Point		0		<input type="text"/>	<input type="text"/>																																																																											
Limit	Equation																																																																																					
0.0	none																																																																																					
Judge Point																																																																																						
0																																																																																						
<input type="text"/>	<input type="text"/>																																																																																					
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>1100</td> </tr> </tbody> </table>		User Range		Lower	Upper	10	1100	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																			
User Range																																																																																						
Lower	Upper																																																																																					
10	1100																																																																																					
Serum Index Limits																																																																																						
	Limit																																																																																					
Hemolysis	0.0																																																																																					
Icterus	0.0																																																																																					
Lipemia	0.0																																																																																					
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>10</td> <td>1100</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	10	1100																																																																															
ValidRange																																																																																						
Lower	Upper																																																																																					
10	1100																																																																																					
<b>Qualitative</b> <input type="radio"/> On <input checked="" type="radio"/> Off																																																																																						

Test Name		Test Code		Measure	Sample Reagent	Ranges Limits	Calibration																																																						
NN	Lpa	IL	Lpa-d																																																										
<b>Calibration</b> <table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>M-point</td> <td></td> <td>5P Log</td> <td></td> </tr> </tbody> </table>		Method		Curve Type		M-point		5P Log		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> </tbody> </table>		Repeats		Stability		2			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> </thead> <tbody> <tr> <td>6</td> <td></td> </tr> </tbody> </table>		No		6		<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>NaCl*</td> <td>0.0</td> </tr> <tr> <td>2</td> <td>TC-Lpa-1</td> <td>**</td> </tr> <tr> <td>3</td> <td>TC-Lpa-2</td> <td>**</td> </tr> <tr> <td>4</td> <td>TC-Lpa-3</td> <td>**</td> </tr> <tr> <td>5</td> <td>TC-Lpa-4</td> <td>**</td> </tr> <tr> <td>6</td> <td>TC-Lpa-5</td> <td>**</td> </tr> </tbody> </table>		Calibrator				Calibrator	Conc	1	NaCl*	0.0	2	TC-Lpa-1	**	3	TC-Lpa-2	**	4	TC-Lpa-3	**	5	TC-Lpa-4	**	6	TC-Lpa-5	**	<b>Condition</b> <input type="checkbox"/> Reagent Blank <input type="checkbox"/> Auto Reagent Blank by Bottle <input type="checkbox"/> Ask for calibration when reagent lot changes		<b>Calibration/R-Blank Limit</b> <table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>		R-Blank Limit		2500.0	mAbs	<input type="text"/>	<input type="text"/>
Method		Curve Type																																																											
M-point		5P Log																																																											
Repeats		Stability																																																											
2			days																																																										
No																																																													
6																																																													
Calibrator																																																													
	Calibrator	Conc																																																											
1	NaCl*	0.0																																																											
2	TC-Lpa-1	**																																																											
3	TC-Lpa-2	**																																																											
4	TC-Lpa-3	**																																																											
5	TC-Lpa-4	**																																																											
6	TC-Lpa-5	**																																																											
R-Blank Limit																																																													
2500.0	mAbs																																																												
<input type="text"/>	<input type="text"/>																																																												
						<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>15.0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>		Cal Reps Range		15.0	%	<input type="text"/>	<input type="text"/>																																																
Cal Reps Range																																																													
15.0	%																																																												
<input type="text"/>	<input type="text"/>																																																												
						<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>200.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>		Min Cal Reps		200.0	mAbs	<input type="text"/>	<input type="text"/>																																																
Min Cal Reps																																																													
200.0	mAbs																																																												
<input type="text"/>	<input type="text"/>																																																												
						<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>35</td> <td>%</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>		Factor Change		35	%	<input type="text"/>	<input type="text"/>																																																
Factor Change																																																													
35	%																																																												
<input type="text"/>	<input type="text"/>																																																												
						<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>		M-Point Curve Fit		0.0	%	<input type="text"/>	<input type="text"/>																																																
M-Point Curve Fit																																																													
0.0	%																																																												
<input type="text"/>	<input type="text"/>																																																												

# Липопротеин (а) 21

ИттипoTD (Иммунотурбидиметрический)



\* -Рекомендуемый параметр.

\*\* - Вводится из паспорта калибратору.

Калибровка многоточечная (6 точек, 0+5 уровней) Spline или 5P Log,  
(калибратор TruCal Lp(a) 21).

Контроль TruLab Lp(a) 21 уровень 1 и 2.