

Ревматоидный фактор. RF

ИммуноTD (Иммунотурбидиметрический)

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **RF** Test Code: IL_ **RF-d** **Measure** Sample Reagent Ranges Limits Calibration

Sample Type
 Serum

Reaction Cycle
☒ Standard ☐ Extended

Reporting Unit
☒ ME/мл* Decimal Points: 1
☐ User Define
 Conv. Factor: 0.000

Methodology
 Type: ☒ End Point ☐ Rate
 Measuring Point: 16 - ☒ 33
 Photometric: ☐ 1 Wavelength ☒ 2 Wavelength
 Primary: 340 Secondary: 750
 Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN **RF** Test Code: IL_ **RF-d** **Measure** **Sample Reagent** Ranges Limits Calibration

Sample Volume **Sampling Condition**

	Sample	Dilution		Diluent	Condition No.	1	2	3	4
	Volume	Sample Vol.	Diluent Vol.						
1	9.0	0.0	0.0		First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
2	3.0	0.0	0.0		Samp. Vol. Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
3	12.0	0.0	0.0		Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
4	0.0	0.0	0.0		Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					Panic H	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					<input checked="" type="checkbox"/> User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
					Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					<input checked="" type="checkbox"/> HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
					<input checked="" type="checkbox"/> ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Reagent Volume
 R1: RF-1 R2: RF-2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	150	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	30	0.0	<input checked="" type="checkbox"/>	20	No Control

Ревматоидный фактор. RF

ИммуноТД (Иммунотурбидиметрический)

Ranges		Limits																																														
<table border="1"><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>0.0</td><td>15</td></tr><tr><td>Female</td><td>0.0</td><td>15</td></tr><tr><td>Other</td><td>0.0</td><td>15</td></tr></tbody></table>		Normal Range				Lower	Upper	Male	0.0	15	Female	0.0	15	Other	0.0	15	<table border="1"><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input type="radio"/> Negative <input checked="" type="radio"/> Positive</td><td><input checked="" type="radio"/> Above <input type="radio"/> Below</td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead><tbody></tbody></table></td><td><table border="1"><thead><tr><td>2600.0</td><td>mAbs</td></tr></thead><tbody></tbody></table></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr></thead><tbody></tbody></table></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td></tr></tbody></table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		100	%	<table border="1"><thead><tr><td>2600.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>	2600.0	mAbs	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr></thead><tbody></tbody></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>		Limit	Equation	0.0	none	<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>		Judge Point		0	
Normal Range																																																
	Lower	Upper																																														
Male	0.0	15																																														
Female	0.0	15																																														
Other	0.0	15																																														
Reaction Slope	Absorbance Limit																																															
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																															
<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		100	%	<table border="1"><thead><tr><td>2600.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>	2600.0	mAbs																																									
Non-Linear Limit																																																
100	%																																															
2600.0	mAbs																																															
<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr></thead><tbody></tbody></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below																																												
Prozone Limit																																																
<input type="radio"/> Above <input checked="" type="radio"/> Below																																																
<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>		Limit	Equation	0.0	none																																											
Limit	Equation																																															
0.0	none																																															
<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>		Judge Point		0																																												
Judge Point																																																
0																																																
<table border="1"><thead><tr><th colspan="2">User Range</th></tr><tr><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>0.0</td><td>500</td></tr></tbody></table>		User Range		Lower	Upper	0.0	500																																									
User Range																																																
Lower	Upper																																															
0.0	500																																															
<table border="1"><thead><tr><th colspan="2">ValidRange</th></tr><tr><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>0.0</td><td>500</td></tr></tbody></table>		ValidRange		Lower	Upper	0.0	500																																									
ValidRange																																																
Lower	Upper																																															
0.0	500																																															
Qualitative																																																
<input type="radio"/> On <input checked="" type="radio"/> Off																																																

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																													
NN	RF	IL	RF-d																																				
Calibration				Condition				Calibration/R-Blank Limit																															
<table border="1"><thead><tr><th colspan="2">Method</th></tr><tr><td>M-point</td><td></td></tr></thead><tbody></tbody></table>				Method		M-point		<table border="1"><thead><tr><th colspan="2">Curve Type</th></tr><tr><td>5P Log</td><td></td></tr></thead><tbody></tbody></table>				Curve Type		5P Log		<table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>				R-Blank Limit		2500.0	mAbs																
Method																																							
M-point																																							
Curve Type																																							
5P Log																																							
R-Blank Limit																																							
2500.0	mAbs																																						
<table border="1"><thead><tr><th colspan="2">Repeats</th></tr><tr><td>2</td><td></td></tr></thead><tbody></tbody></table>				Repeats		2		<table border="1"><thead><tr><th colspan="2">Stability</th></tr><tr><td></td><td>days</td></tr></thead><tbody></tbody></table>				Stability			days	<table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>15.0</td><td>%</td></tr></thead><tbody></tbody></table>				Cal Reps Range		15.0	%																
Repeats																																							
2																																							
Stability																																							
	days																																						
Cal Reps Range																																							
15.0	%																																						
<table border="1"><thead><tr><th colspan="2">No</th></tr><tr><td>6</td><td></td></tr></thead><tbody></tbody></table>				No		6		<table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>100.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>				Min Cal Reps		100.0	mAbs	<table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>35</td><td>%</td></tr></thead><tbody></tbody></table>				Factor Change		35	%																
No																																							
6																																							
Min Cal Reps																																							
100.0	mAbs																																						
Factor Change																																							
35	%																																						
<table border="1"><thead><tr><th colspan="3">Calibrator</th></tr><tr><th></th><th>Calibrator</th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>NaCl*</td><td>0.0</td></tr><tr><td>2</td><td>TC-RF-1</td><td>**</td></tr><tr><td>3</td><td>TC-RF-2</td><td>**</td></tr><tr><td>4</td><td>TC-RF-3</td><td>**</td></tr><tr><td>5</td><td>TC-RF-4</td><td>**</td></tr><tr><td>6</td><td>TC-RF-5</td><td>**</td></tr></tbody></table>				Calibrator				Calibrator	Conc	1	NaCl*	0.0	2	TC-RF-1	**	3	TC-RF-2	**	4	TC-RF-3	**	5	TC-RF-4	**	6	TC-RF-5	**	<table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead><tbody></tbody></table>				M-Point Curve Fit		0.0	%				
Calibrator																																							
	Calibrator	Conc																																					
1	NaCl*	0.0																																					
2	TC-RF-1	**																																					
3	TC-RF-2	**																																					
4	TC-RF-3	**																																					
5	TC-RF-4	**																																					
6	TC-RF-5	**																																					
M-Point Curve Fit																																							
0.0	%																																						
<table border="1"><tbody><tr><td><input type="checkbox"/></td><td>Reagent Blank</td></tr><tr><td><input type="checkbox"/></td><td>Auto Reagent Blank by Bottle</td></tr><tr><td><input type="checkbox"/></td><td>Ask for calibration when reagent lot changes</td></tr></tbody></table>				<input type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes																														
<input type="checkbox"/>	Reagent Blank																																						
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																						
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																						

Ревматоидный фактор. RF

ИттипoTD (Иммунотурбидиметрический)



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору

Калибровка многоточечная (6 точек, 0+5 уровней, Spline или 5P Log)

калибратор TruCal RF

Контроль TruLab Protein уровень 1 и 2.