

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN TP Test Code: IL_ TP-d Measure Sample Reagent Ranges Limits Calibration

Sample Type: Serum

Reaction Cycle: Standard Extended

Reporting Unit: г/л * Decimal Points: 1

User Define: Conv.Factor: 0.000

Methodology: Type: End Point Rate Measuring Point: 16 - 33 Photometric: 1 Wavelength 2 Wavelength Primary: 546 Secondary: 700

Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN TP Test Code: IL_ TP-d Measure Sample Reagent Ranges Limits Calibration

Sample Volume: Sample Dilution Diluent Diluent Warning Limit Tests

	Sample Volume	Dilution Sample Vol.	Dilution Diluent Vol.
1	3.25	0.0	0.0
2	2.0	0.0	0.0
3	6.5	0.0	0.0
4	0.0	0.0	0.0

Reagent Volume: R1 TP-d1 R2 TP-d2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	160	0.0	✓	20	No Control
R2	40	0.0	✓	20	No Control

Sampling Condition:

Condition No.	1	2	3	4
First Run	✓			
Samp.Vol.Reduction		✓		
Below N-Range				
Above N-Range				
Panic L				
Panic H		✓		
User Range L				
User Range H	✓	✓		
Noise				
Prozone				
HIGH!	✓	✓		
ABS!	✓	✓		

Параметры для ввода в программу анализатора iLab-Tourus

Test Name: NN TP Test Code: IL_ TP-d Measure Sample Reagent Ranges Limits Calibration

Sample Type: Serum

Reaction Cycle: Standard Extended

Reporting Unit: г/л * Decimal Points: 1

User Define: Conv.Factor: 0.000

Methodology: Type: End Point Rate Measuring Point: 33 Photometric: 1 Wavelength 2 Wavelength Primary: 546 Secondary: 700

Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN TP Test Code: IL_ TP-d Measure Sample Reagent Ranges Limits Calibration

Sample Volume: Sample Dilution Diluent Diluent Warning Limit: 0 Tests

	Sample Volume	Dilution	Diluent
1	4.0	0.0	0.0
2	3.0	0.0	0.0
3	5.0	0.0	0.0
4			

Reagent Volume: R1: TP-d R2:

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	200	0.0	✓	20	No Control
R2					

Sampling Condition:

Condition No.	1	2	3	4
First Run	✓			
Samp.Vol.Reduction		✓		
Below N-Range				
Above N-Range				
Panic L				
Panic H		✓		
User Range L				
User Range H	✓	✓		
Noise				
Prozone				
HIGH!	✓	✓		
ABS!	✓	✓		

Общий белок

Biuret (биуретовый) общие параметры

Ranges		Limits																																														
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>66</td> <td>88</td> </tr> <tr> <td>Female</td> <td>66</td> <td>88</td> </tr> <tr> <td>Other</td> <td>66</td> <td>88</td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.5</td> <td>150</td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.5</td> <td>150</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	66	88	Female	66	88	Other	66	88	User Range		Lower	Upper	0.5	150	ValidRange		Lower	Upper	0.5	150	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> Negative</td> <td><input checked="" type="radio"/> Positive</td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>0</td> <td>%</td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Reaction Slope		<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	Non-Linear Limit		0	%	Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0
Normal Range																																																
	Lower	Upper																																														
Male	66	88																																														
Female	66	88																																														
Other	66	88																																														
User Range																																																
Lower	Upper																																															
0.5	150																																															
ValidRange																																																
Lower	Upper																																															
0.5	150																																															
Reaction Slope																																																
<input type="radio"/> Negative	<input checked="" type="radio"/> Positive																																															
Non-Linear Limit																																																
0	%																																															
Serum Index Limits																																																
	Limit																																															
Hemolysis	0.0																																															
Icterus	0.0																																															
Lipemia	0.0																																															
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>		Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off	<table border="1"> <thead> <tr> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td><input checked="" type="radio"/> Above</td> <td><input type="radio"/> Below</td> </tr> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> <tr> <td>Limit: 0.0</td> <td>Equation: none</td> </tr> <tr> <td colspan="2">Judge Point: 0</td> </tr> </tbody> </table>		Absorbance Limit		<input checked="" type="radio"/> Above	<input type="radio"/> Below	2500.0	mAbs	Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	Limit: 0.0	Equation: none	Judge Point: 0																												
Qualitative																																																
<input type="radio"/> On	<input checked="" type="radio"/> Off																																															
Absorbance Limit																																																
<input checked="" type="radio"/> Above	<input type="radio"/> Below																																															
2500.0	mAbs																																															
Prozone Limit																																																
<input type="radio"/> Above	<input checked="" type="radio"/> Below																																															
Limit: 0.0	Equation: none																																															
Judge Point: 0																																																

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																																														
NN	TP	IL	TP-d																																																																					
Calibration				Condition				Calibration/R-Blank Limit																																																																
<table border="1"> <thead> <tr> <th colspan="2">Method</th> </tr> </thead> <tbody> <tr> <td>1-point</td> <td></td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>Linear</td> <td></td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">Repeats</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td></td> <td>days</td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCalU</td> <td>**</td> </tr> <tr> <td>2</td> <td></td> <td></td> </tr> <tr> <td>3</td> <td></td> <td></td> </tr> <tr> <td>4</td> <td></td> <td></td> </tr> <tr> <td>5</td> <td></td> <td></td> </tr> </tbody> </table>				Method		1-point		Curve Type		Linear		Repeats		2		Stability			days	No				Calibrator				Calibrator	Conc	1	TruCalU	**	2			3			4			5			<input checked="" type="checkbox"/> Reagent Blank <input type="checkbox"/> Auto Reagent Blank by Bottle <input type="checkbox"/> Ask for calibration when reagent lot changes				<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> </thead> <tbody> <tr> <td>20.0</td> <td>%</td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> </thead> <tbody> <tr> <td>50.0</td> <td>mAbs</td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> </thead> <tbody> <tr> <td>20</td> <td>%</td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>%</td> </tr> </tbody> </table>				R-Blank Limit		2500.0	mAbs	Cal Reps Range		20.0	%	Min Cal Reps		50.0	mAbs	Factor Change		20	%	M-Point Curve Fit		0.0	%
Method																																																																								
1-point																																																																								
Curve Type																																																																								
Linear																																																																								
Repeats																																																																								
2																																																																								
Stability																																																																								
	days																																																																							
No																																																																								
Calibrator																																																																								
	Calibrator	Conc																																																																						
1	TruCalU	**																																																																						
2																																																																								
3																																																																								
4																																																																								
5																																																																								
R-Blank Limit																																																																								
2500.0	mAbs																																																																							
Cal Reps Range																																																																								
20.0	%																																																																							
Min Cal Reps																																																																								
50.0	mAbs																																																																							
Factor Change																																																																								
20	%																																																																							
M-Point Curve Fit																																																																								
0.0	%																																																																							

Общий белок

Biuret (биуретовый)



* -Рекомендуемый параметр.

** - Вводится из паспорта к калибратору

Тип калибровки: Линейная, калибратор TruCal U или по стандарту из набора.
Контроль по TruLab N и TruLab P.